

A complete manual is available at [www.sixcase.com](http://www.sixcase.com)

**Installation materials:**




- Washers (4x)
- M8 Nuts (4x)
- Nylon spacers (4x)
- Wall plug 10mm (4x)
- Dowel screw (4x)

Suitable for installation on brick/concrete or wood


**IMPORTANT:**

- Minimum required wall thickness: 3mm
- Install on a flat surface
- Always use the spacers
- Recommended height: 1.5 – 1.6m upper holes (above ground)
- Install in shadow
- Place adapter indoors

**Installation**

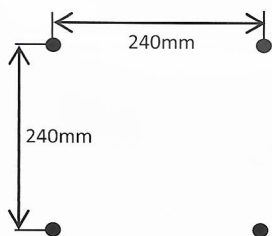

- 1: Drill 10mm holes (4x)
- 2: Insert Plugs and dowel screws (20mm protrusion)
- 3: Place spacers
- 4: Feeder cable: Drill a 12 mm hole through the wall

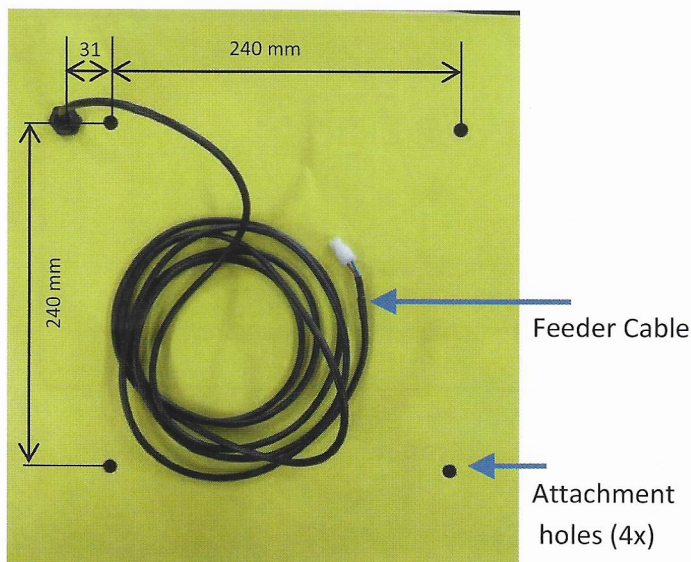

5: place washers and M8 Nuts on the inside

