Resusci Junior

ENGLISH	Directions for Use
DEUTSCH	Gebrauchsanweisung
FRANÇAIS	Mode d'emploi
ESPAÑOL	Instrucciones de utilización
ITALIANO	Istruzioni per l'Uso
NEDERLANDS	Handleiding
NORSK	Bruksanvisning
SVENSKA	Bruksanvisning
SUOMI	Käyttöohje

Your new Resusci Junior

Thank you for purchasing the Laerdal Resusci Junior. We know you will find this manikin to be a reliable and long serving addition to your training equipment. You can have confidence in your Laerdal Resusci Junior; after all Laerdal has been developing and manufacturing training manikins for over 50 years. During this time Laerdal has remained the world-wide market leader.

As with all your training equipment it will perform better and have a longer life span if it is maintained properly. Laerdal manikins are renowned world-wide for their reliability.

If you have any questions or comments on the Laerdal Resusci Junior please contact your local Laerdal representative who will be happy to assist you.

This product is in compliance with the essential requirements of Council Directive 89 / 336 / EEC, EMC Directive.

Warranty:

Please see the Laerdal Global Warranty or visit www.laerdal.com

WEEE

Waste Electrical and Electronic Equipment

This appliance is marked according to the European directive 2002/96/EC on Waste Electrical and Electronic Equipment (WEEE). By ensuring this product is disposed of correctly, you will help prevent potential negative consequences for the environment and human health, which could otherwise be caused by inappropriate waste handling of this product.

The symbol  on the product, or on the documents accompanying the product, indicates that this appliance may not be treated as household waste. Instead it shall be handed over to the applicable collection point for the recycling of electrical and electronic equipment.

Disposal must be carried out in accordance with local environmental regulations for waste disposal.

For more detailed information about treatment, recovery and recycling of this product, please contact your local city office, your household waste disposal service or the Laerdal representative where you purchased the product.

Unpacking and assembly

If the manikin is stored in the carrying case it is necessary to assemble the manikin before use.

Place the torso on a firm surface.

Pull out the hip bolt.

Push the two legs into their seats.

Fasten legs to the body by inserting the hip bolt through the opening in the body shell, through the holes in the thighs and out through body shell opening on the opposite side.

Press the bolt into its seat to avoid loosening it by accident.

Put the shorts and jacket on the manikin.

Training mat

All versions of Resusci Junior contain a training mat. This has two purposes:

- Comfort and convenience for students and instructors when practicing BLS skills on the Resusci Junior.
- Movement of the assembled Resusci Junior around the classroom / training venue

To use in water

If the manikin is to be used for training in water, see the section on "Water rescue" page 6.

Manikin with SkillGuide

If you have purchased the Resusci Junior with SkillGuide:

- Remove the SkillGuide protective cover.
- Plug the SkillGuide cable into the connector on the right side of the manikin.
- Remove the paper tab coming from the battery compartment on the back of the SkillGuide.
- Press the "ON" button to begin practice.
- When transporting the manikin using the training mat it is recommended that the SkillGuide be unplugged and moved separately.

Repacking after use

If the manikin has been used in water, see instructions in the section on "Water rescue" page 6.

To store in hard case

If the SkillGuide is attached, unplug the cable and store the SkillGuide in the soft pouch.

Take off the shorts. Detach the legs from the body in the following way:

Extract the hip bolt as follows:

- Exert pressure on the protruding end of the bolt. Continue to push to get the hip bolt further out
- Grasp and pull out the hip bolt

When the legs have been removed, press the bolt into its seat to avoid loosening it by accident. Put the torso in the case, face up. Place a leg on each side of the manikin.

Remove dirt from the training mat by wiping it with a damp cloth. Put the training mat into the plastic bag provided, and place on top of the manikin. This will prevent dirt from the training mat getting on the manikin.

SkillGuide

Consistent and objective feedback on performance during practice helps develop and reinforce skills. The SkillGuide provides feedback to the instructor and student to support the student's initial skill development by means of a colour light display.

Inflation

When lung inflation volume reaches 200 ml the green signal light will be activated. If lung inflation volume exceeds 300 ml, an orange warning light is activated. A red warning light indicates an inflation that is too rapid.

Chest compression

When the breast bone area of the chest is depressed by a minimum of 50 mm, the green signal light is activated.

Incorrect hand position

The sensor is located under the switch cover mounted on the chest plate. The red warning light will be activated if the chest is compressed with hands placed outside the recommended area. Providing the downward pressure is exerted on the lower half of the sternum the red light will not activate. However, if downward pressure is exerted on the xiphoid sternum, on the upper half of the sternum or on the chest wall away from the sternum the red light will activate.

To use

See Unpacking page 3. The SkillGuide will turn "OFF" automatically if no activity takes place for 5 minutes.

Note: if the SkillGuide is turned "ON" and the red signal light for wrong hand position comes on the SkillGuide is not completely plugged into the manikin.

Power supply

The four 1.5V alkaline batteries, size C, will provide approximately 125 hours of classroom use. This will vary depending on user patterns and the type of batteries.

Connection cable

This is permanently connected to the SkillGuide and has an 8 pin male connector which is to be connected to the manikin. The connector has been moulded for proper orientation prior to plugging it into the manikin's side.


Caution

Make sure the SkillGuide is unplugged from manikin:

- When airway is changed, or
- When internal service is carried out.


	Light	Indication	Limits ¹
Inflation volume:	green orange	correct too much	200ml 350ml
Compression depth:	green	correct	≥50mm
Wrong hand position:	red	Pressure outside of acceptable area	
Too fast inflation	red		

1. Measurement tolerance in the manikin +/- 15% in normal room temperature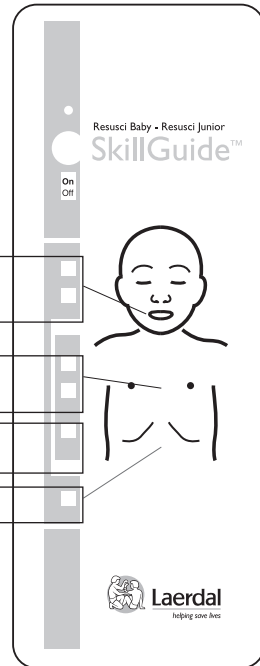

Disposable Junior Airway

The Junior Airway is designed for disposal after class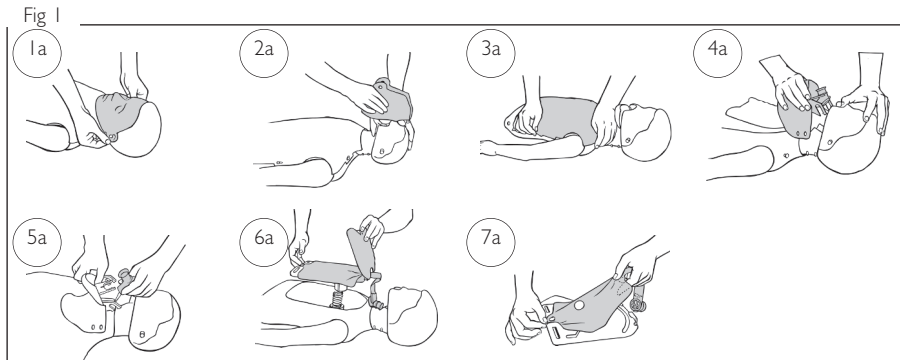


To remove (Fig. 1)

1. Detach the manikin face at each ear.
2. Lift manikin face off valve coupling.
3. Remove chest skin.
4. Lift off jaw from posts in head.
5. Remove valve.
6. Lift off rib plate.
7. Remove used airway and discard.

To install (Fig. 2)

1. Attach lung to hook and lay lung flat on chest plate.
2. Pull valve through rib plate. Attach rib plate.
3. Snap valve onto jaw.
4. Reattach jaw to posts in head.
5. Fasten chest skin onto its retainers.
6. Push face coupling into airway valve.
7. Fasten face onto retainers at each ear.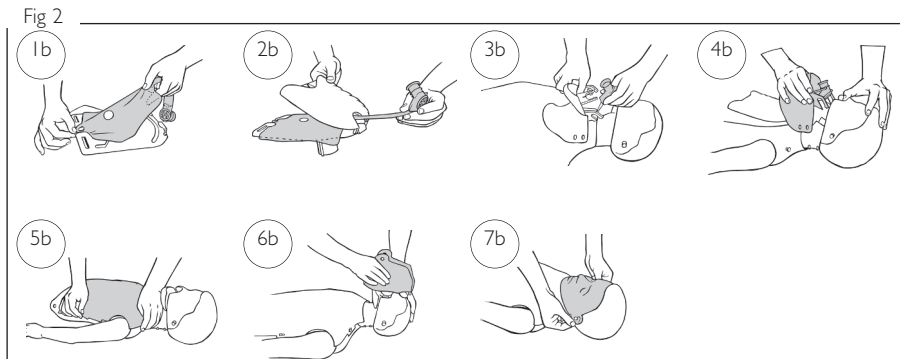

Sanitation

Sanitation of Manikin Faces, see separate sheet.

General manikin cleaning

Routine disinfection of the entire manikin is not required, but periodic cleaning of the head, torso, chest skin and training mat is recommended. These should be washed with a warm soapy solution, and then rinsed with a clean damp cloth.

Proper care will maintain the appearance and general condition of your Resusci Junior. Hand or machine wash clothes with soap or laundry detergent in water 40°C. Iron with warm iron. May also be dry cleaned.

Water rescue


The Resusci Junior manikin is designed for training in water resuscitation and rescue techniques. If you have purchased a water-rescue version of the Resusci Junior it is ready to use. An optional Water Rescue Kit (184010) is available to make the manikin simulate, a semi-submerged floating, unconscious victim, or a victim that will sink to the bottom of the pool. The manikin may be used in salt or fresh water.

Caution:

The SkillGuide version of Resusci Junior must not be used for water rescue.


Installing the Water Rescue Kit (184010)

Mount the floatation device as follows: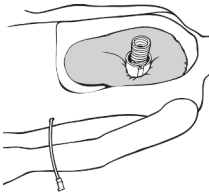

Unbutton the chest skin and lift out the chest and lung plate assembly. Place the floatation device underneath the lung plate over the compression cylinder. Push the tubing out through the manikin's side.

Mount the weights as follows:

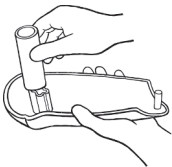Undo the four screws of each thigh. Place the weight over the upper cylinder piece, one weight in each thigh. Reassemble the thigh.

Using Resusci Junior for water rescue

Now the Water Rescue Kit has been installed, the device can be inflated or deflated without any disassembly of the manikin.

Preparing to float:

Attach the inflation side of the pump bulb to the tubing and squeeze. It requires about 18-22 full squeezes of the pump to inflate the device. Pinch the tubing, remove the pump bulb, and insert the stopper plug.

When the manikin is in the water; you can open the tube to evacuate air until the position is correct, i.e. the angle between the water and body is 30°.


Note: After use, the floatation device is deflated by attaching the deflation side of the pump to the tubing and squeezing. When all the air is removed from the floatation device, the pump bulb will not recover from your squeeze. At this point, remove the pump bulb and replace the stopper plug.

Repacking after use

After use in water, the manikin should be well dried before being returned to storage. Removing the manikin from the water by the arms will help the manikin drain quickly. To further promote gravity drainage, prop the manikin upright for a few minutes. Use a cloth to dry the exterior and interior of the manikin.

The following exemplifies uses of the Resusci Junior manikin in water rescue training programs and lifesaving competition events.

In-water resuscitation

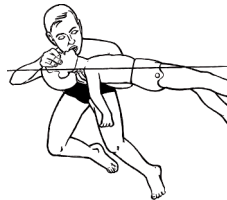
Rescue breathing may be practised while the manikin is in the water.

Recognition of drowning

When the floatation device has been inflated, the manikin will float face down, semi-submerged in the water. This is a position typical of an unconscious person.

Surfacing and towing

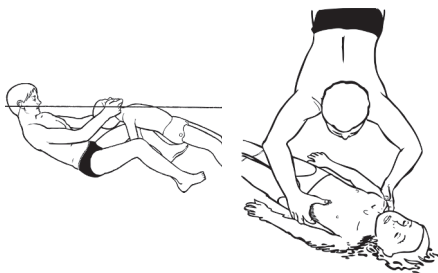When the floatation is not used, the manikin will sink. Thus, the manikin can be used for training rescue diving and surfacing as well as realistic towing and landing techniques.

If the manikin is dropped on its back it will normally turn over; and sink face down to the bottom. Water is allowed to enter through the neck opening and through special holes in the trunk and extremities.

Caution:

Make sure the Resusci Junior manikin is not unintentionally released in deep water without the floatation device in place. The manikin will sink to the bottom, which might cause inconvenient rescue operations.

Consumables

151201xx	Manikin Face Shields (6 rolls)
152400	Manikin Wipes (pkg.50)
152401	Manikin Wipes (pkg.1200)
020310	Valve complete (pkg.50)
183010	Junior face (pkg.6)
183210	Airway complete (pkg.25)
183211	Airway complete (pkg.100)
183220	Filter and lung (pkg. 100)

Optional equipment

184010	Water Rescue kit
184600	Choking kit
184610	Junior Choking objects (pkg. 5)

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