


SIXCASE.COM

Ready to improve?

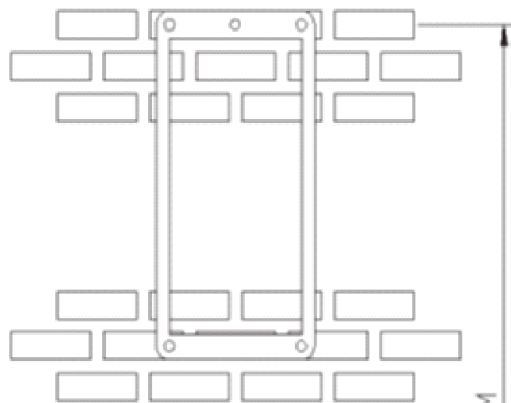

Use the supplied template to mark the holes. For further instructions, please check the manual.

Mounting on brick or concrete: Place the supplied template against the wall and mark the holes. Drill the round 10 mm holes to a minimum depth of 60 mm, to ensure there is sufficient depth for the plug. We recommend drilling the upper holes at between 1.5 and 1.6 metres from the ground. The holes should always be drilled in the middle of the brick.



Mounting on wood: Drill small holes of approximately 4 mm diameter in the wood and, using a tool, screw the wiretap or screws firmly into the wood and then mount the SixCase as instructed above.

SC1400 series only: The wiretap should extend not more than 40 mm from the wooden surface.

